



LETS/SPEAK PLUS AUGMENTATIVE COMMUNICATION DEVICES WITH LEVELS INSTRUCTION SHEET

Congratulations on your purchase of the Lets/Speak Plus Entry-Level Communicator. The LETS/SPEAK PLUS IS A MULTI-LEVEL AUGMENTATIVE COMMUNICATOR TO OFFER HIGH QUALITY, RELIABLE PERFORMANCE, AS WELL AS MANY OTHER IMPORTANT FEATURES FOR A VERY AFFORDABLE PRICE. THE LETS/SPEAK PLUS IS MANUFACTURED TO THE HIGHEST COMMERCIAL STANDARDS TO PROVIDE MANY YEARS OF RELIABLE OPERATION. WE FULLY GUARANTEE THE LETS/SPEAK PLUS OPERATION FOR A FULL YEAR, INCLUDING PARTS AND LABOR.*

Distinctive Features of the Lets/Speak Plus

- THE LETS/SPEAK 6 PLUS HAS 6 MESSAGE PER LEVEL AND 2 CORE MESSAGES.
- THE LETS/SPEAK 24 PLUS HAS 24 MESSAGE PER LEVEL AND 4 CORE MESSAGES.
- 10 LEVELS OF COMMUNICATION.
- 10 SECONDS OF RECORDING TIME PER MESSAGE.
- ALLOWS YOU TO PROGRAM, RECORD AND PLAYBACK 248 INDEPENDENT MESSAGES.
- RECORD, ERASE, LOCK AND UNLOCK CONTROLS ARE RECESSED ON THE BACK OF THE DEVICE.
- VOLUME UP/DOWN AND LEVELS UP/DOWN ON THE FRONT OF THE DEVICE FOR THOSE USERS THAT CAN MAKE THERE OWN CHANGES FOR MORE INDEPENDENCE.
- RECORD AND LEVELS UP/DOWN CAN ALSO BE LOCKED OUT FOR UNWANTED CHANGES.
- THE LETS/SPEAK PLUS USES STANDARD LETTER SIZE PAPER. NO CUTTING IS NEEDED TO SPEED UP CREATING YOUR OVERLAYS, JUST **PRINT-N GO**.
- A HANDLE IS SUPPLIED FOR EASE OF TRANSPORTING AS WELL AS A CARRYING STRAP.
- WATER RESISTANT FOR EASY CLEANING.
- DROP RESISTANT, ABLE TO WITHSTAND ROUGH HANDLING.
- THE SHATTERPROOF CASE IS MADE OF A HIGH IMPACT PLASTIC.
- LIGHT WEIGHT AND PORTABLE (LESS THAN 2 LBS.).

*SEE WARRANTY AGREEMENT FOR DETAILS.

RECORD OPERATION:

Note: It is important that the built-in microphone located on the front right-hand side of the Lets/Speak 6 Plus is open and clear during recording. It is recommended the room you record in is quiet. Any background noise in the room will be recorded with your voice. Have the Lets/Speak Plus on a flat surface close to you. Speak directly toward the built-in microphone on the lower right hand corner of the Lets/Speak Plus.

The core messages only need to be recorded once on any level. They will stay the same for every level.

Recording procedure:

1. Slide the Power switch located on the back of the Lets/Speak Plus to the ON position.
2. Press the Level Up or Down button to set the level you want to record on indicated by the LED's on the front of the Lets/Speak Plus on the right side.
3. Press the Red recording button on the back of the Lets/Speak Plus. A beep will be heard and the recording LED on the top panel on the left side.
4. Press the message cell that you want to record your message on.
5. Begin speaking your message. When you are finished speaking your message press the message cell again to stop recording.
6. To record another message cell you can press that cell and start speaking your message. Again, when you are finished speaking your message press the cell again to stop the recording.
7. When you are finishing recording all your messages press the Red recording button again.

Level Erase procedure:

The Level Erase will only erase the messages for the level that is selected when performing this function.

1. Turn the Lets/Speak Plus On by sliding the switch on the back of the Lets/Speak Plus to the ON position.
2. Press the Level Up or Down button on the front of the Lets/Speak Plus to get to the level you want to erase.
3. Press the Red record button on the back of the Lets/Speak Plus.
4. Press and hold in the Erase button on the back of the Lets/Speak Plus. A beep will be heard.
5. Release the erase button. The level is now erased.

Level Lock procedure:

The Level Lock will disable the Level Up and Down buttons on the front of the Lets/Speak Plus.

1. Turn the Lets/Speak Plus ON by sliding the switch on the back of the Lets/Speak Plus to the ON position.
2. Hold the Red record button in for about 10 seconds.
3. Press either Level Up or Down button on the front of the Lets/Speak Plus.
4. The LED for that level will start to Flash.
5. Press the Lock button on the back of the Lets/Speak Plus.
6. The Level Up and Down buttons are now disabled.

Record Lock procedure:

The Record button Lock will disable the Record button on the back of the Lets/Speak Plus.

1. Turn the Lets/Speak Plus ON by sliding the switch on the back of the Lets/Speak Plus to the ON position.
2. Press the Lock button on the back of the Lets/Speak Plus.
3. The Record button is now disabled.

Level Unlock procedure:

The Level Unlock will enable the Level Up and Down buttons on the front of the Lets/Speak Plus.

1. Turn the Lets/Speak On by sliding the switch on the back of the Lets/Speak Plus to the ON position.
2. Hold the Red record button in for about 10 seconds.
3. Press either Level Up or Down button on the front of the Lets/Speak Plus.
4. The LED for that level will start to Flash.
5. Press the Unlock button on the back of the Lets/Speak Plus.
6. Press the Unlock button on the back of the Lets/Speak Plus.
7. The Level Up and Down buttons are now enabled.

Record unlock procedure:

The Record unlock will enable the Record button on the back of the Lets/Speak Plus.

1. Turn the Lets/Speak Plus ON by sliding the switch on the back of the Lets/Speak Plus to the ON position.
2. Press the Unlock button on the back of the Lets/Speak Plus.
3. The Record button is now enabled.

OVERLAY DESIGN AND LAYOUT:

Graphic overlays (picture cards) can be changed easily by simply sliding the overlay through the slot on the top of the housing grid. There are several methods you can use to make overlays for the Lets/Speak Plus. Options include:

1. Drawing or printing corresponding icons or words on a sheet of standard Letter size paper. Section each icon so that it appears in each of the message cell cutouts on the top grid.
2. Developing your custom overlay using the Mayer-Johnson Boardmaker software. Blank templates are available on the on our website @ www.amdi.net.

CLEANING THE Lets/Speak Plus

Note: DO NOT SUBMURGE THE LETS/SPEAK PLUS IN WATER OR ALLOW WATER TO ENTER THE INTERNAL SPEAKER HOLES... THE LETS/SPEAK 24 PLUS IS WATER RESISTANT – NOT WATER PROOF.

1. Turn the Lets/Speak Plus OFF.
2. Remove the overlay from the grid.
3. With a wet wash cloth and mild soap wipe the soiled areas clean.
4. To rinse, use a damp sponge and wipe the away the soapy area.
5. The grid can easily be removed for cleaning by unscrewing the 10 Phillips head screws on the back of the Lets/Speak Plus.

Care and Maintenance

The Lets/Speak Plus is designed to automatically power down after a message is not played for more then 30 seconds. However, the power down mode is still an active mode and will drain some power from the battery, affecting battery life. Therefore, if the Lets/Speak Plus is not going to be used for a long period of time (2 to 3 hours) it is recommended that the unit be turned off.

If the Lets/Speak Plus should fail to operate, or operates incorrectly, follow these steps:

1. Check that the batteries are good.
2. Check that the Lets/Speak Plus is ON and a Message has been recorded at that level.
3. Make sure that there are no obstacles blocking the message membrane square.
4. Turn the Lets/Speak Plus OFF for 15 Seconds, then back ON and retry playback.
5. For further assistance, feel free to contact our technical support team at 1-888-353-AMDI (2634).

95 SHERWOOD AVE., FARMINGDALE, NY 11735
PHONE: (631) 531-8888 FAX: (631) 531-8811

1-888-353-AMDI
HTTP://WWW.AMDI.NET